



# Step by Step Guide Oral Fluid Twist Test

## 1. Remove cap

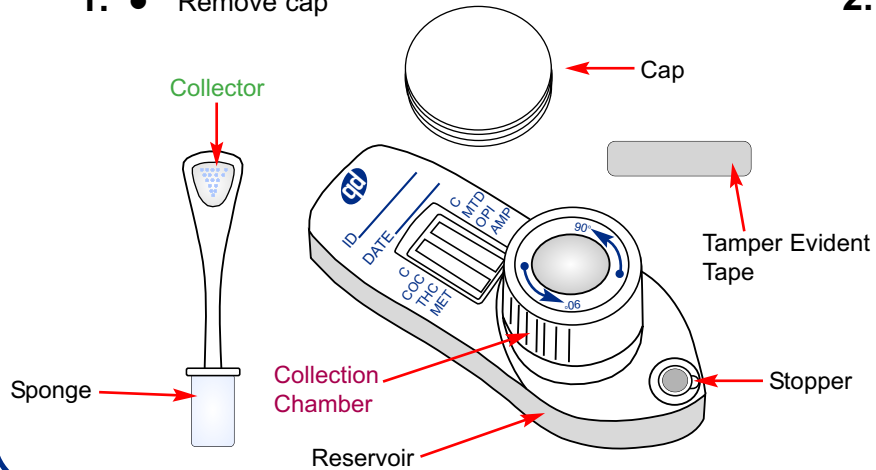


- ## 2.
- Nil by Mouth, for 10 minutes before test.
  - Actively swab the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press the sponge between the tongue and teeth.
  - When no hard spots can be detected, saturate sponge with oral fluid which will take approximately 2 minutes .

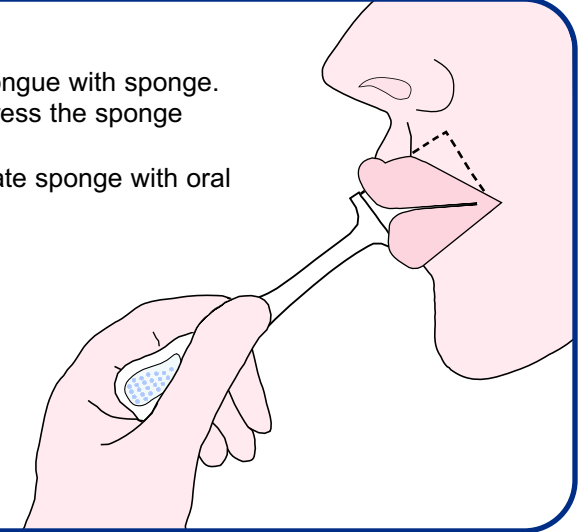



## TEST WILL NOT RUN WITH INADEQUATE SAMPLE VOLUME

- ## 3.
- With the test device on a flat surface insert the collector with the notches aligned.
  - Twist collector 90° until locked in place.
  - Wait for 1 minute.

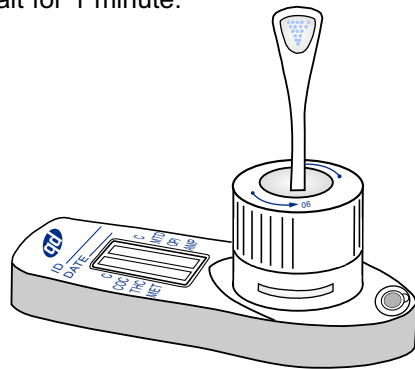


- ## 4.
- Turn collection chamber counter-clockwise.
  - After a further 9 minutes, read results.


Remember:  
Turn collection chamber

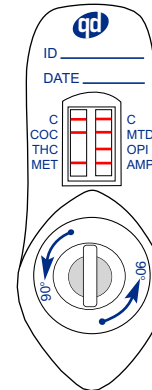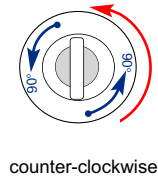


- ## 5.
- If results need confirmation, remove collector and secure cap back onto collection chamber with Quantum Labs security seal.
  - Secure stopper with tamper evident tape.
  - Fill in Quantum Labs Chain of Custody form.
  - Send to Quantum Laboratories for confirmation.

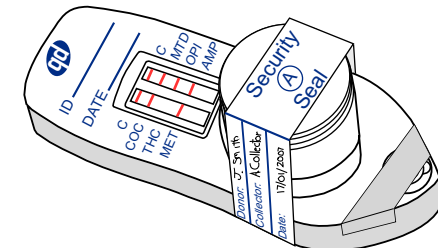


**THERE IS NO MEANING ATTRIBUTED TO LINE INTENSITY OR WIDTH  
ANY EVIDENCE OF A TEST LINE SHOULD BE CONSIDERED A NEGATIVE RESULT**

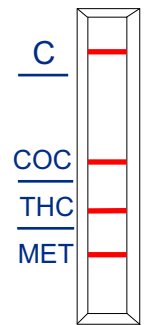


**NEGATIVE**  
*(all drugs)*

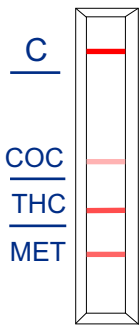


**NEGATIVE**  
*(all drugs)*

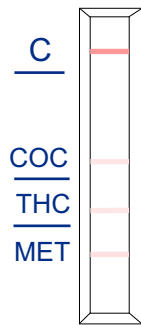


**NEGATIVE**  
*(all drugs)*

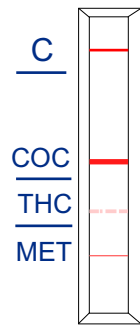


**NEGATIVE**  
*(all drugs)*

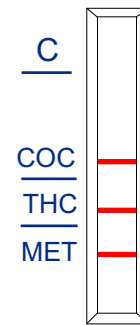


**INVALID**  
*re-test*

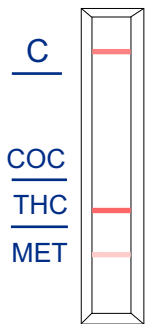


**POSITIVE**  
*(COC)*

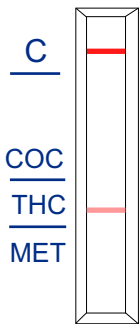


**POSITIVE**  
*(COC / MET)*

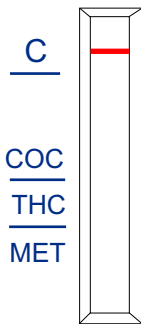

**POSITIVE**  
*(all drugs)*

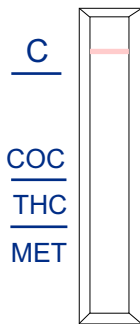

**POSITIVE**  
*(all drugs)*

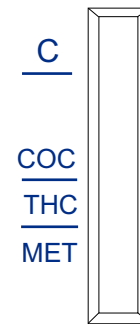

**INVALID**  
*re-test*

	<u>Cut-off Detection Times</u> ng/ml (approximate)	
<b>AMP</b> amphetamine	50	12 Hours
<b>BZO</b> benzodiazepines	10	6-48 Hours
<b>COC</b> cocaine	50	24 Hours
<b>MET</b> methamphetamine	20	24-48 Hours
<b>MTD</b> methadone	30	24 Hours
<b>OPI</b> opiates	40	6-12 Hours
<b>PCP</b> phencyclidine	10	24 Hours
<b>THC</b> thc / cannabis	100	24-48 Hours Single use: 12 hours