




Adulteration Test Step by Step Guide


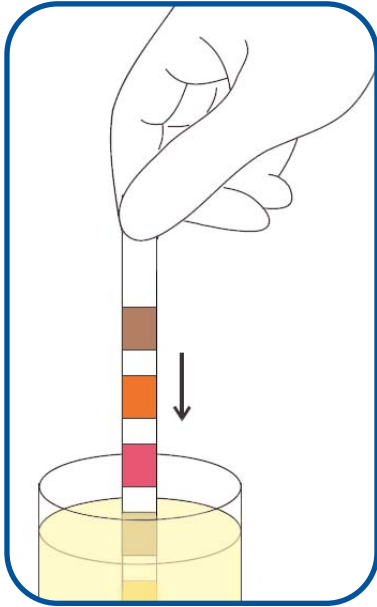


Allow the strip, urine specimen, and/or controls to reach room temperature (15-30°C) prior to testing.

1. Remove the strip from the closed canister and use it as soon as possible. Immediately close the canister tightly after removing the required number of strip(s).

Completely immerse the reagent areas of the strip in fresh, well-mixed urine and immediately remove the strip out of the urine specimen to avoid dissolving the reagents.

2. Hold the strip in a horizontal position and immediately bring the edge of the strip into contact with an absorbent material (e.g. a paper towel) to avoid mixing chemicals from adjacent reagent areas and/or soiling hands with urine.

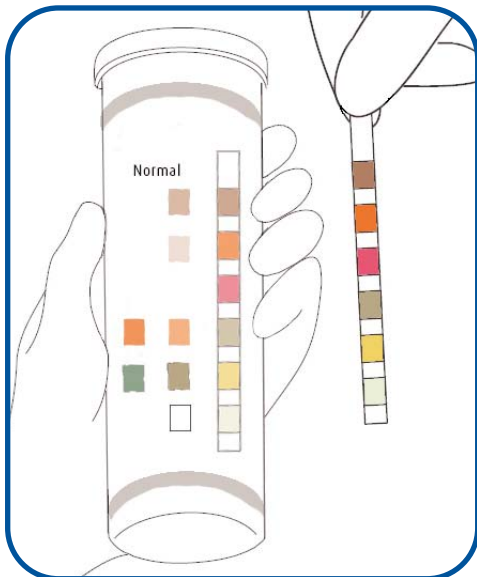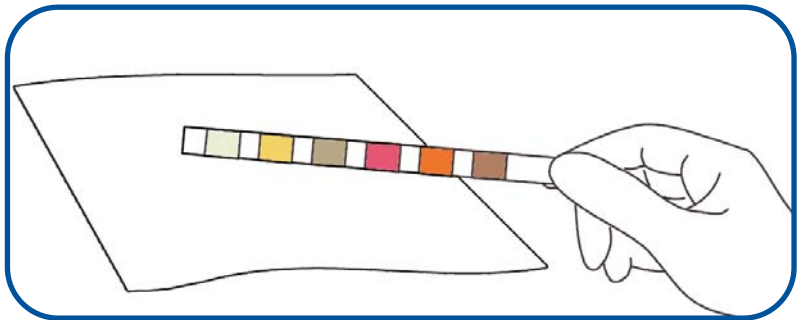

3. Read results in 1 minute by comparing the reagent areas to the corresponding colour blocks on the canister label. Hold the strip close to the colour blocks and match carefully.

Do not interpret results after 4 minutes.